

SmartMan Skills Refresher Station


NEW 2015

SmartMan Pro Training continues to set the standard for CPR training. We recognize that there are situations where you do not require a fully certified trainer. Students need to brush up on their skills periodically.

SmartMan provides the perfect solution. It can be set up as an independent student learning station.

Skills Adjustment on a Regular Basis

The SmartMan Learning Station gives the student objective feedback as they perform so they can modify their skills as required.

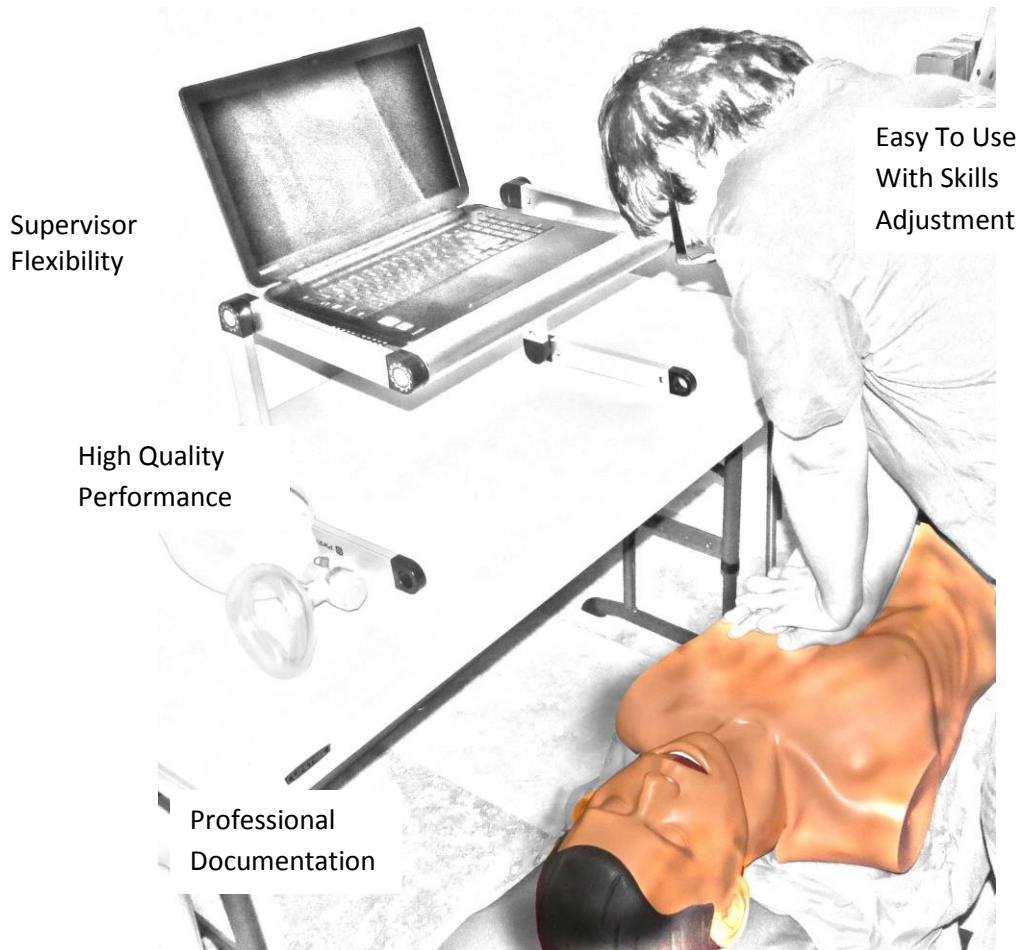

Can Be Used For Multiple Sites

STUDENT SELF LOGIN

The trainer decides how to organize the station logins names:


- By Student Name
- By Group Name with Student Name
- By Class Name with Student Name

Student Procedures

1. Click on their name and enter password.
2. Click on the Skills Activity.
3. Click on Start and perform skills

Students see accurate feedback as they perform and are able to adjust their performance to improve their skills.

When they finish the skill they immediately see if they have attained the target score. If a student feels like practicing again, he or she simply clicks on the Restart Button. Students can come back to practice whenever they have a few minutes of spare time.

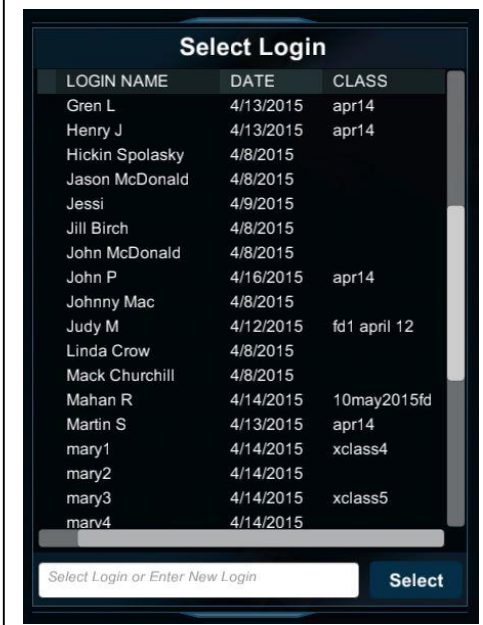

LOGIN NAME	DATE	CLASS
Gren L	4/13/2015	apr14
Henry J	4/13/2015	apr14
Hickin Spolasky	4/8/2015	
Jason McDonald	4/8/2015	
Jessi	4/9/2015	
Jill Birch	4/8/2015	
John McDonald	4/8/2015	
John P	4/16/2015	apr14
Johnny Mac	4/8/2015	
Judy M	4/12/2015	fd1 april 12
Linda Crow	4/8/2015	
Mack Churchill	4/8/2015	
Mahan R	4/14/2015	10may2015fd
Martin S	4/13/2015	apr14
mary1	4/14/2015	xclass4
mary2	4/14/2015	
mary3	4/14/2015	xclass5
marv4	4/14/2015	


Students can even do a self-test with or without feedback.

It is a quick efficient way to make sure CPR skills are always at their best.

[More Information](#)

SUPERVISOR ACCESS TO RESULTS

The supervisor has full access to student results; it is possible to examine the results of a whole class or of an individual. It is possible to sort by class, name, date and skill performed.

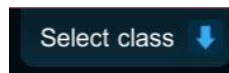

The computer does not need to be connected to the manikin to create classes or to examine results. Once every 90 days compare results or print a class list with results.

[More Information](#)



COORDINATE MANY SITES

Set Achievement Target

Set a target achievement level and a date by which it has to be done.

Set The Time Interval


Set a time. For example have students demonstrate competence every 90 days or every 180 days.

Quick Zip and Send Results

Enter the name, email address and click on the "Email Zipped File" Button. All results are sent to your inbox.

The zipped file contains all of the records of the skills performed.

You have full access to all information to print class lists with scores, produce charts and compare results.

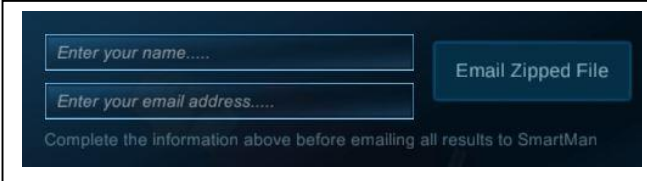

Enter your name.....

Enter your email address.....

Email Zipped File


Complete the information above before emailing all results to SmartMan

[More Information](#)

PROFESSIONALLY DISPLAY RESULTS

SmartMan results information is in an easy to read format so it can be quickly displayed. Select the Class, click ALL Results for that class and click on the Export and View Button. The file will open directly in Excel or OpenCalc.

- Quickly Print a class list with names, dates and scores.
- Or turn the list into a chart
- Or sort and then create a chart in ascending order.

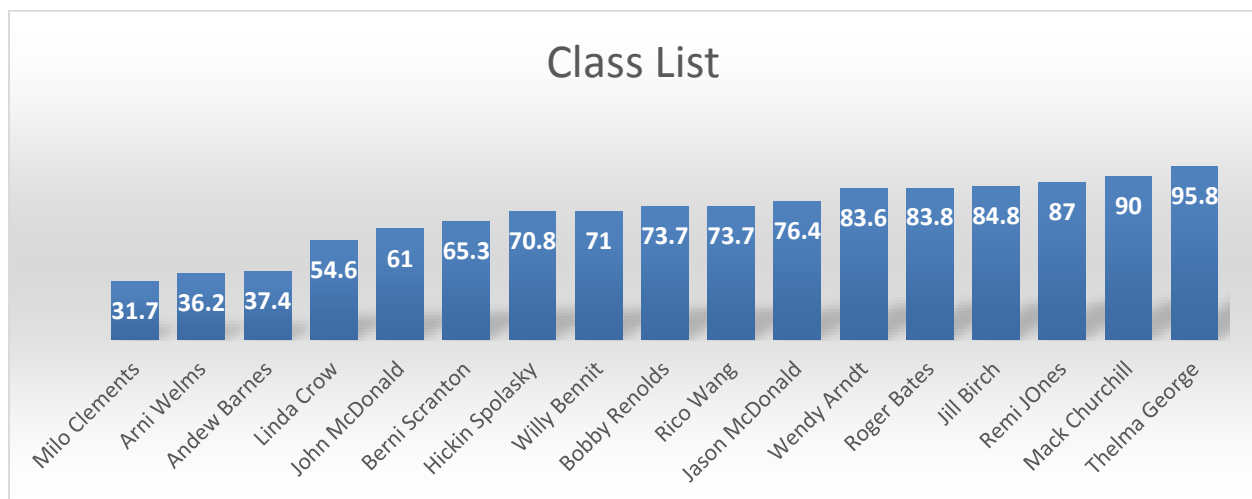

[More Information](#)

The SmartMan Refresher Station
For Quick Honing of Skills When You Really Do Not Need
A Fully Certified Instructor Present



16155 Sierra Lakes Pkwy Ave, Ste 160 – 20, Fontana, CA 92336
Tel: 877 774-4323 Fax: 707 929-0079 Email: purchasing@ambusmartman.com