

SmartMan Results

SmartMan saves the results of all Skills Menu Practices and Tests

Results are an accurate analysis of the results of every practice or test performed on SmartMan. It is possible to simply look at the color of the bars and the overall score to quick a quick idea of how well a person has done.

Alternatively, you may wish to explore in detail, how various sub skills were performed. SmartMan gives you both options.

It is also possible to view the overall trends of groups of results with the Admin Module.

The accurate data can also be exported from the Admin module so that detailed statistical analysis can be performed.

View The Results

View Immediately: As soon as a practice or a test is completed, SmartMan provides a detailed report on the skills that were performed.

View at a Later Time: Results are stored on the local machine under the login of each individual who performed a skill. A person can log in at a later time and View Previous Results.

Understanding The Results

See the section on results for each skill area for detail on the colored bars, the scores, and the detail. These are in the section on Results for each skill.

- 5. Results 1. Compressions
- 5. Results 2. Ventilations
- 5. Results 3. CPR
- 5. Results 4. Advanced Airway

Printing Results

Results are an accurate analysis of the results of every practice or test performed on SmartMan.

To print results you must be logged in and you must be viewing results. You view results either immediately after performing a set of skills or by going to the Actions Menu and selecting 'View Previous Results.

Once you have results showing on the screen you can print the results.

Print Screen of Results: When the results are showing press the Print Screen button in the middle of the screen. This will capture the information on the current screen and can be used to make a complete record of the color bars and details of the performance.

Pressing the print screen button will take a screen capture of the current screen. It will capture only the SmartMan screen. When you press the button several things will happen:

1. It will create a *.png file.
2. This will be automatically saved in the Smartman directory where the program is located.
3. The default picture viewer will open with the picture. This is likely "Windows Photo Viewer". Click on File, then click on Make a copy and type a file name and select a location to save the file.

To capture different results, for example cycle two, in SmartMan click on the next cycle button and then press screen capture again and repeat the above procedure.

Print Results: When the results are showing press the Print Button in the middle on the far right side of the screen.

This will generate a report with the person's login name, the type of activity performed and then detail on the overall performance along with more detail on several individual sub skills.

This report is provided in an html format. As it is a form and in html, it must have Active-X in your computer set to on in order to fill the form with the data. You can then print or copy the results

Results Into Word: An html form cannot be copied and pasted into Word. It will copy the form but not the data. First save the html as an MHT, MHTML, MAFF or other archive file format. Then open that file and copy it. Then paste it into Word.

Admin Module: This is a separate utility that provides for trend analysis of results data that has been submitted to the server.

Access to this data is restricted to you key codes and is password protected.

This module is also used to export data in a comma delimited file so that the data can be used in other statistical analysis programs.